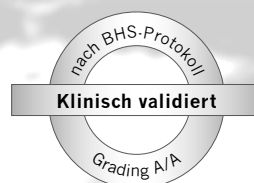
boso

BOSCH + SOHN
GERMANY

boso TM-2430 PC 2

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The result of sophisticated research and development of blood pressure measuring instruments to ensure highest precision and reliability is the boso TM-2430 PC 2. This ambulatory blood pressure monitor (ABPM) has been the first and only ABPM since 2003 manufactured in line-production applied on the International Space Station ISS. After the first successful testing in 2002 several projects have been accomplished meanwhile with the boso TM-2430 PC 2 to examine the effects of zero gravity to the cardio-vascular system.

In a press conference the European Space Agency (ESA) stressed the importance of the results gained from these studies. Although the conditions in space are different from every-day use on earth, the high quality products of boso have proven their reliability and precision.

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Your patients need convenience.

The boso TM-2430 PC 2 is the lightweight amongst the ambulatory blood pressure monitors. Its low weight and compact dimensions stand for pleasant carrying convenience and best possible acceptance by your patients.

You need accuracy.

The oscillometric detection method guarantees accurate blood pressure readings in almost every situation. The boso TM-2430 PC 2 stores up to 350 measurements, is easy to handle and quickly programmable, with fully adjustable intervals and a user-friendly sleep function. The unit is clinical validated according to BHS-Protocol Grading A for Systole and Diastole.

You need efficiency.

Your boso TM-2430 PC 2 will be amortized very quickly, because of its low cost and the powerful evaluation software. With the PMM software you can also transfer and analyse the readings from your patients' digital blood pressure meter for a reliable and effective long term monitoring of therapy.

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The complete set includes:
case, battery charger, two sets of rechargeable batteries, carrying pouch, belt, wipe clean adult-sized cuff, PC connection cable and the software boso profile-manager.

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Easy programming.

You enter the settings with the PC-keyboard. You fix the intervals, beginning and ending of the sleep period or you choose the preset automatic interval. The unit can be initialized via a fixed patient denomination.

Fast and reliable evaluation of the patients' data.

Just plug-in and go. With the boso profile-manager you make perfect diagnosis in just a few seconds. It enables you to create tables of pressure readings, display data either as longterm profiles and calculate mean arterial pressure, pulse and morning blood pressure surge (MBPS) and perform other statistical analyses on your data.

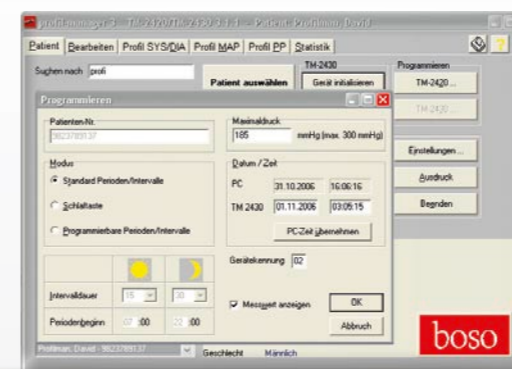
Clear documentation of readings.

All evaluations can be printed as coloured tables and graphics – for a perfect documentation and as an additional record to your existing patients' data base.

Interface to all common patient information systems.

The QMS-certified GDT-Interface allows an easy integration to all common patient information systems.

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

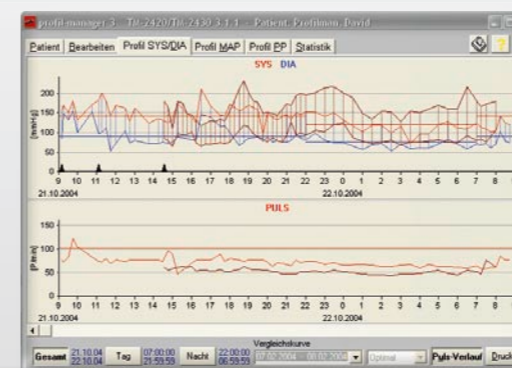
Possibility of presenting two curves (SYS, DIA, PULS, MAP, PP) at the same time (success of therapy can be recognized easily)

Gesamt		Tag		Nacht	
Messwerte	48	Messwerte	17	Messwerte	17
	Min. Mittel. Max. StdAbw. > Limit		Min. Mittel. Max. StdAbw. > Limit		Min. Mittel. Max. StdAbw. > Limit
SYS [mmHg]	70 137,2 209 27,2 50,8%	SYS [mmHg]	92 144,3 209 25,3 50,0%	SYS [mmHg]	70 117,1 151 22,5 35,3%
DIA [mmHg]	51 93,4 153 24,1 30,8%	DIA [mmHg]	51 90,7 153 25,1 33,3%	DIA [mmHg]	51 99,2 96 11,4 22,5%
PULS [1/sec]	46 71,7 120 11,2 3,1%	PULS [1/sec]	46 74,3 120 11,7 4,2%	PULS [1/sec]	50 64,4 73 4,5 0,0%
MAP [mmHg]	61 101,3 166 23,3 41,5%	MAP [mmHg]	73 107,2 166 23,1 37,5%	MAP [mmHg]	61 84,5 114 14,4 35,3%
PP [mmHg]	13 53,8 100 20,4 96,9%	PP [mmHg]	16 55,5 100 22,0 62,5%	PP [mmHg]	13 48,9 79 14,5 64,7%
Tag / Nacht Abweichung		Nacht			
MBPS	22,8 mmHg				

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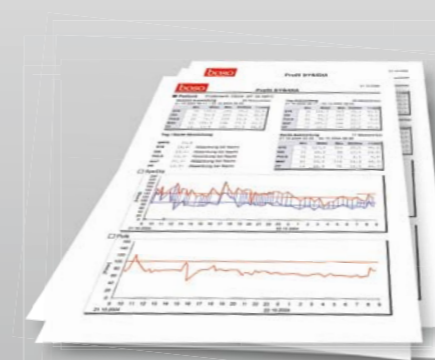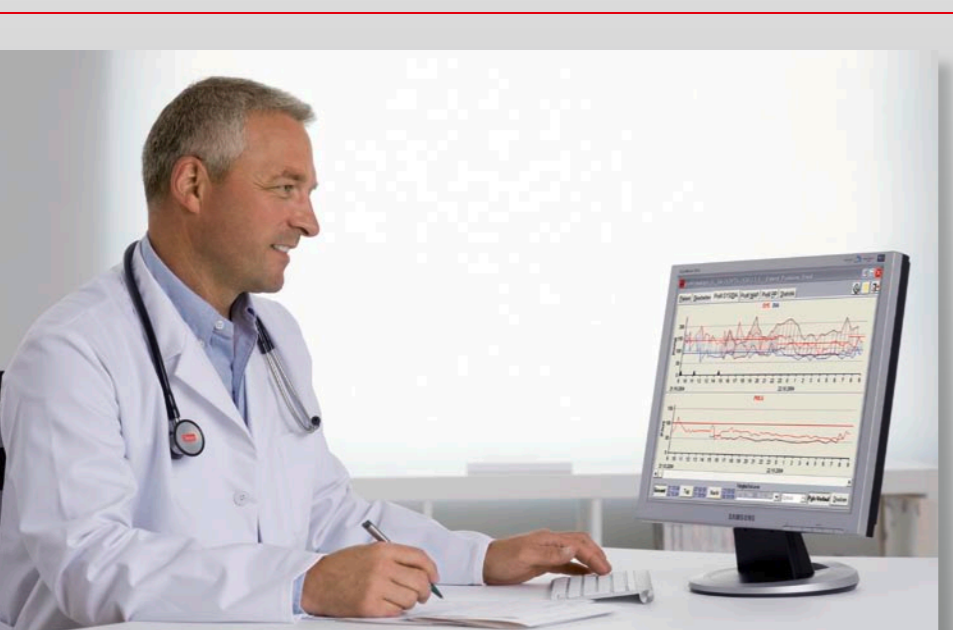Statistic evaluation incl. calculation of morning blood pressure surge

Dokumentation von Langzeitprofil und Statistik
Documentation of long-term profiles and statistics

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The boso PMM-System – for best therapy of your hypertension patients

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PMM stands for profile-manager-monitoring.
And it works very easy:

1. You get extensive measurements data as a base for a safe diagnosis, throughout the 24-hour blood pressure reading.
2. After the 24-hour reading with the boso TM-2430 PC 2, the monitoring of therapy will be continued by the patients with the boso digital blood pressure meters with PC interface. These are equipped with a measurement memory of 325 respectively 280 readings. The patients take the self-measurements at home and the boso blood pressure device stores all the important data like date, time, blood pressure and pulse.
3. With the profile-manager-software all readings will be transferred to the PC – for an effective and reliable monitoring of therapy.

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What's your benefit as a physician?

boso blood pressure devices with PMM system provide optimal compliance through an effective and reliable monitoring of therapy, in addition to the ambulatory blood pressure measurement with the boso TM-2430 PC 2. You will get perfectly registered data from your patient, which can be analysed and printed on your PC immediately.

The ambulatory blood pressure monitor boso TM-2430 PC 2, as well as the digital blood pressure instruments boso medilife PC 3 and boso medicus PC 2 run with the same software – saving expense and allowing an easier use and evaluation with the PC. In addition to this it is no problem, because of the built-in GDT/BDT interface, to integrate the PMM system into your patient information system.

What's your patients' benefit?

The patients measure comfortably at home, mistakes as a result of wrong notation will be impossible. The compliance will be improved. The extensive monitoring of therapy ensures an effective and gentle medication.

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BOSCH + SOHN GmbH & Co. KG